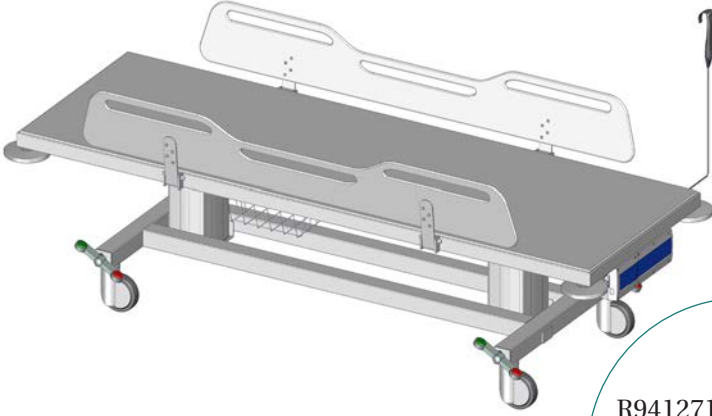
R9412717 - R9412721
R9432717 - R9432721
R9452717 - R9452721
R9462717 - R9462721

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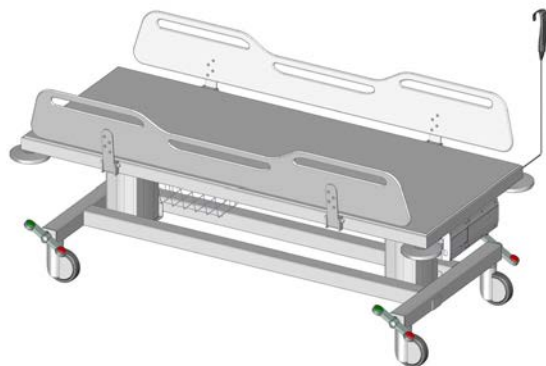
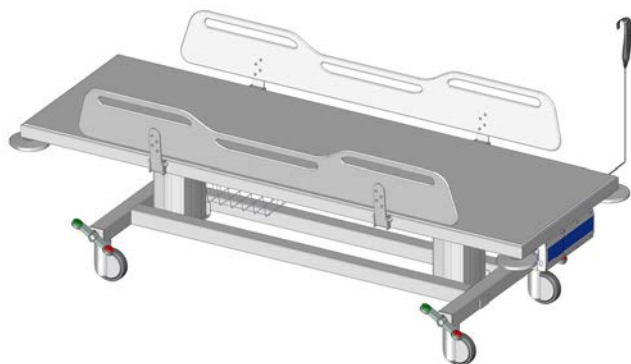
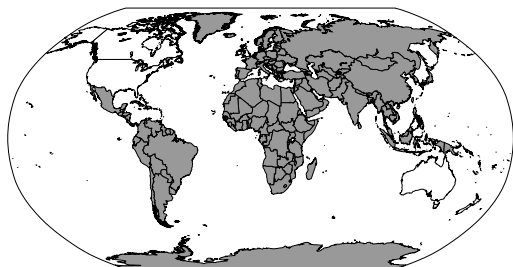
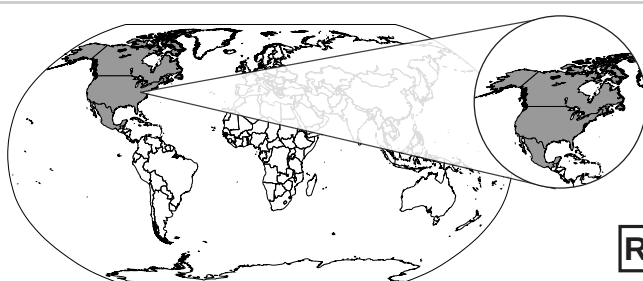
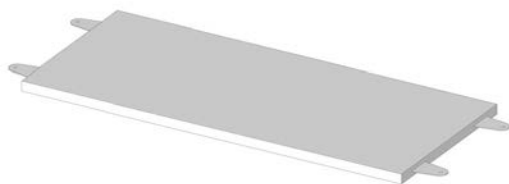
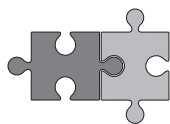
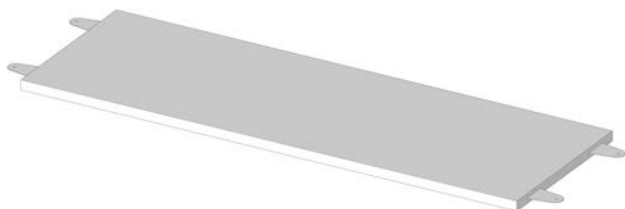
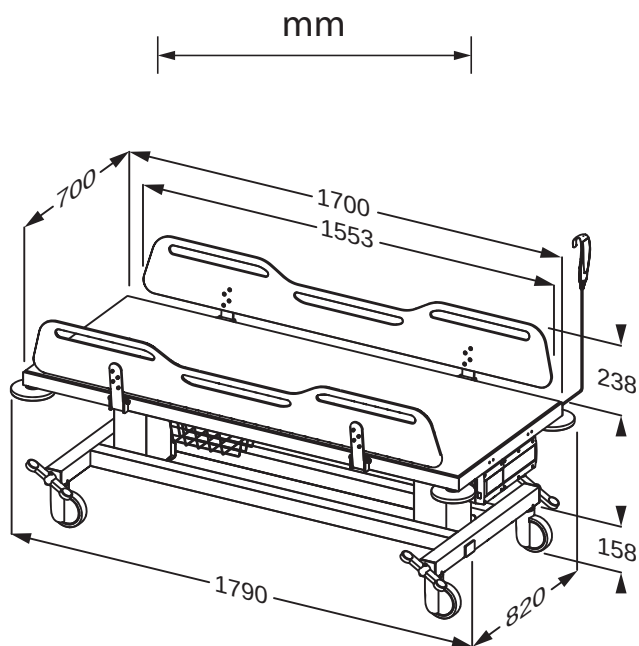
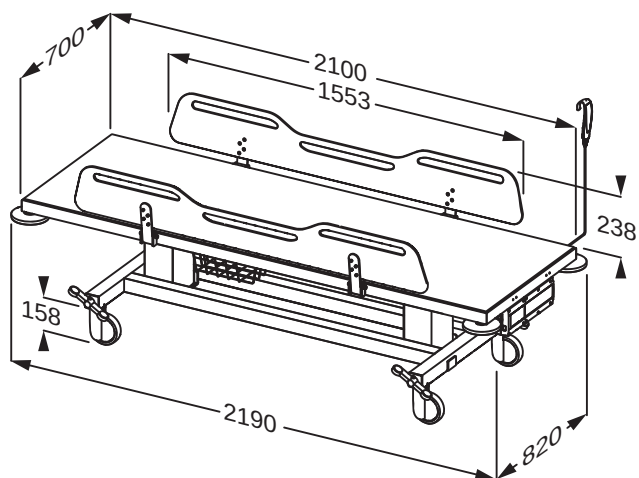
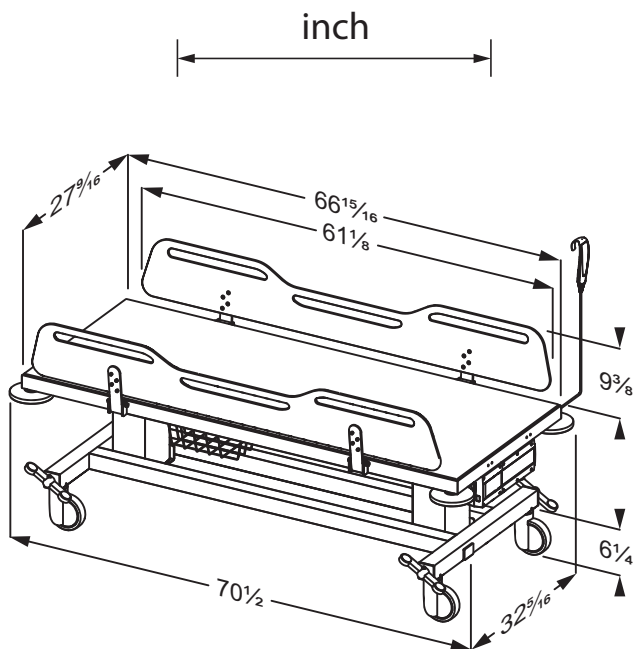
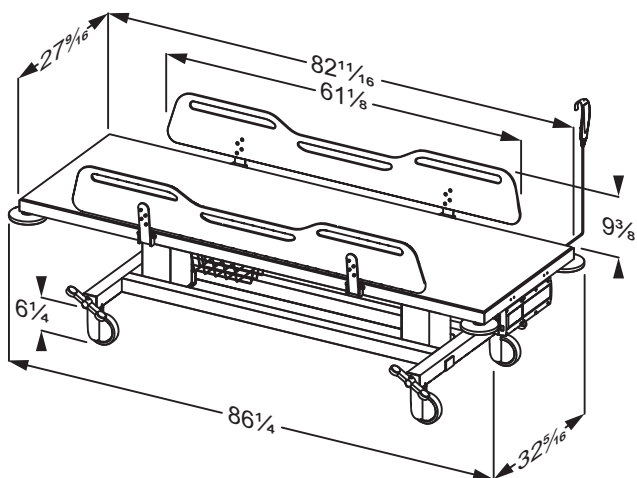
R9432721, R9452721, R9412721, R9462721

Content

Symbols used in this manual	9
General safety	10
Product label	11
Intended use	13
Intended operator profile	13
Intended user profile	13
Operating environment	13
Contraindications for use	13
Biocompatibility	13
Operating instructions	14
Safety brakes	14
Side supports	15
Height adjustment	16
Resetting	16
Charging the battery	17
Accessories	19
Mattress	19
Cleaning	20
Electromagnetic compatibility	21
Electromagnetic immunity	21
Electromagnetic emissions	21
Technical data	22
Trouble-shooting	23
Storage and transportation	24
Disposal and recycling	24

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Head injury warning

Feet injury warning

Electricity or potentially dangerous voltage.

Compatible products

Do not leave the patient unattended.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

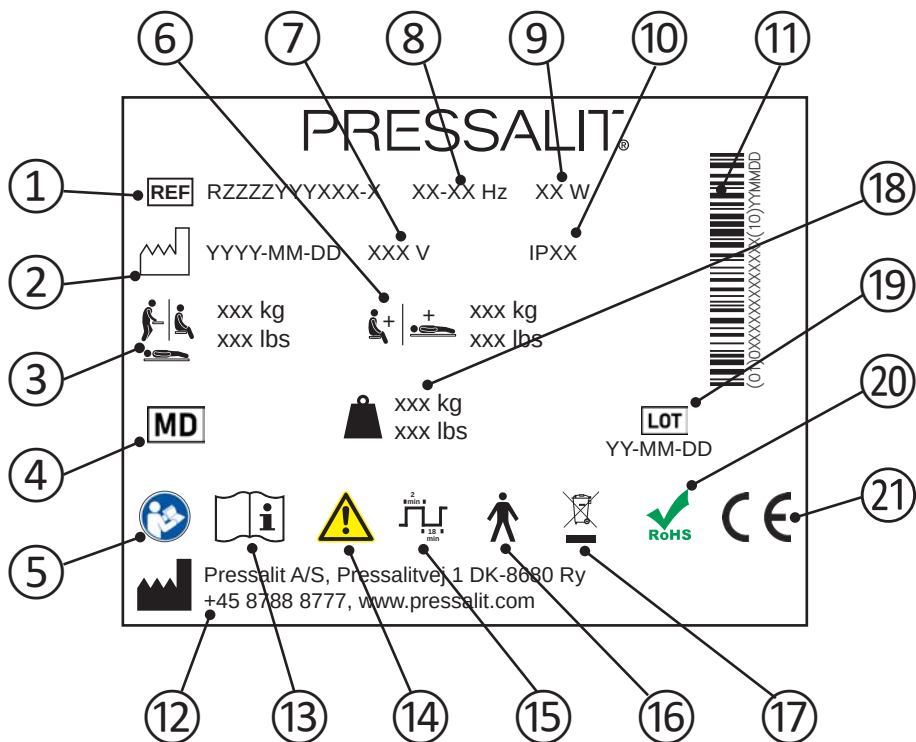
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

en

- ① Article number
- ② Production date
- ③ Maximum user weight
- ④ Medical device
- ⑤ Consult manual for use
- ⑥ Max load
- ⑦ Mains voltage
- ⑧ Frequency

- en
-
- ⑨ Maximum power consumption
-
- ⑩ IP class
-
- ⑪ GS1-128 barcode
-
- ⑫ Name and address of manufacturer
-
- ⑬ Consult manual for use
-
- ⑭ Consult the manual for important cautionary information such as warnings and precautions
-
- ⑮ Duty cycle
-
- ⑯ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock
-
- ⑰ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
-
- ⑱ Product weight
-
- ⑲ Batch code
-
- ⑳ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
-
- ㉑ This product is CE-marked in accordance with the European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC. A declaration of conformity can be found at www.pressalit.com.

Intended use

This product is intended for changing a person who is unable to stand or sit. The person is lying horizontally on the change trolley.

Intended operator profile

The product is to be used by personnel trained in health care and trained in changing a person in their care.

Intended user profile

The product can be used by all - adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in public changing places, schools, private homes, nursing homes and hospitals under normal working conditions.

Contraindications for use

Use of the change trolley is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the change trolley only after consulting with the attending physician.

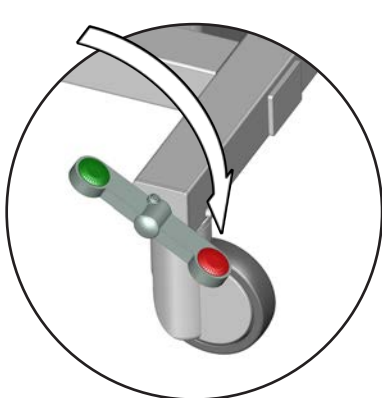
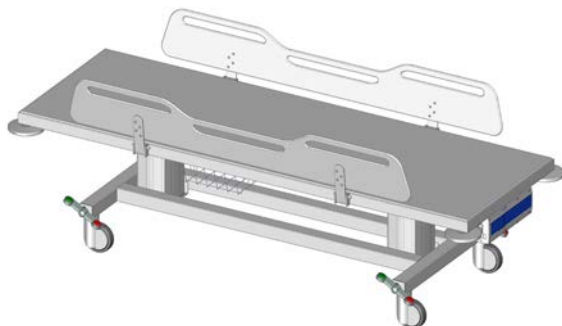
Biocompatibility

The materials in contact with the skin of the patient or the user during the changing procedure, have been tested for their content of harmful substances according to ISO 10993 Biocompatibility, and are therefore considered suitable for this purpose.


Operating instructions

Safety brakes

The change trolley is equipped with safety brakes on all four castor wheels to ensure that it will not move unintentionally.

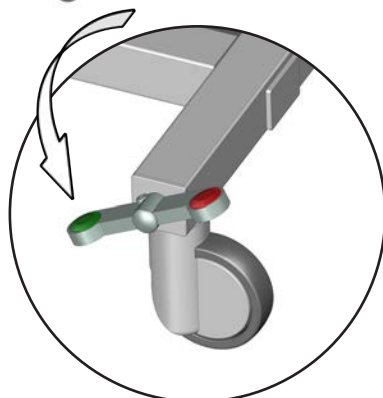To lock the brakes: Step on the red button on one of the four rocker pedals to lock the brakes on all four wheels.

To unlock the brakes: Step on the green button on one of the four rocker pedals to unlock the brakes on all four wheels.

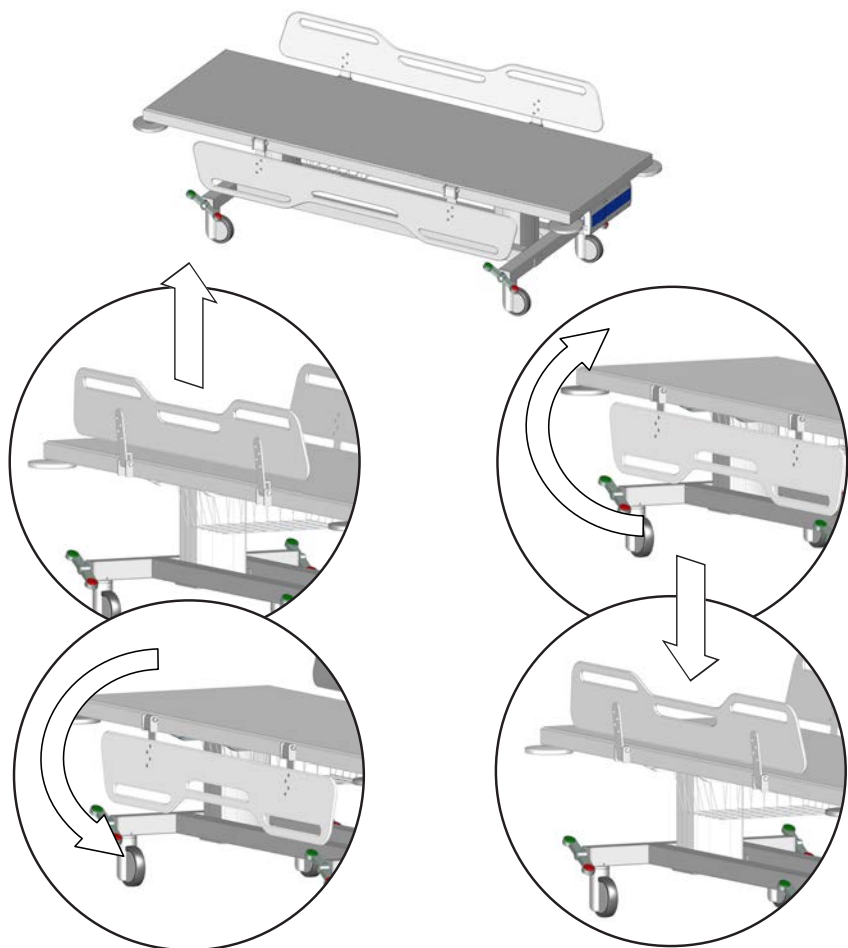
Always ensure that the brakes are locked when transferring a patient to and from the change trolley.

Do not leave the patient unattended.

Side supports

The side supports can be folded up and down individually for easy transfer of the patient.


To fold down: Lift the side support upwards to unlock it, and swing it down.

To fold up: Swing the side support up and push it down to lock it in place.

When the trolley is in use, make sure that all side supports are erect and in locked position to protect the patient from falling out of the unit.

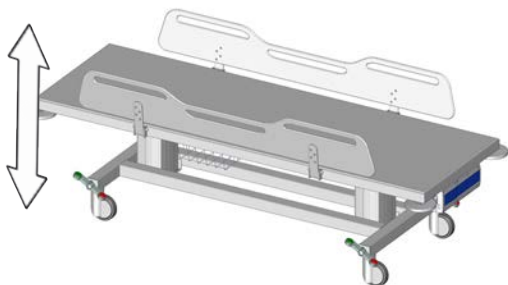Height adjustment

The change trolley can be adjusted 400 mm / 15¾" up and down.

Max user weight:

200 kg / 440 lbs

Max load:

250 kg / 550 lbs

The height adjustment is controlled by the wired remote control unit.

Press and hold the Up button to raise the unit.
When the button is released, the motion stops.

Press and hold the Down button to lower the unit.
When the button is released, the motion stops.

To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.

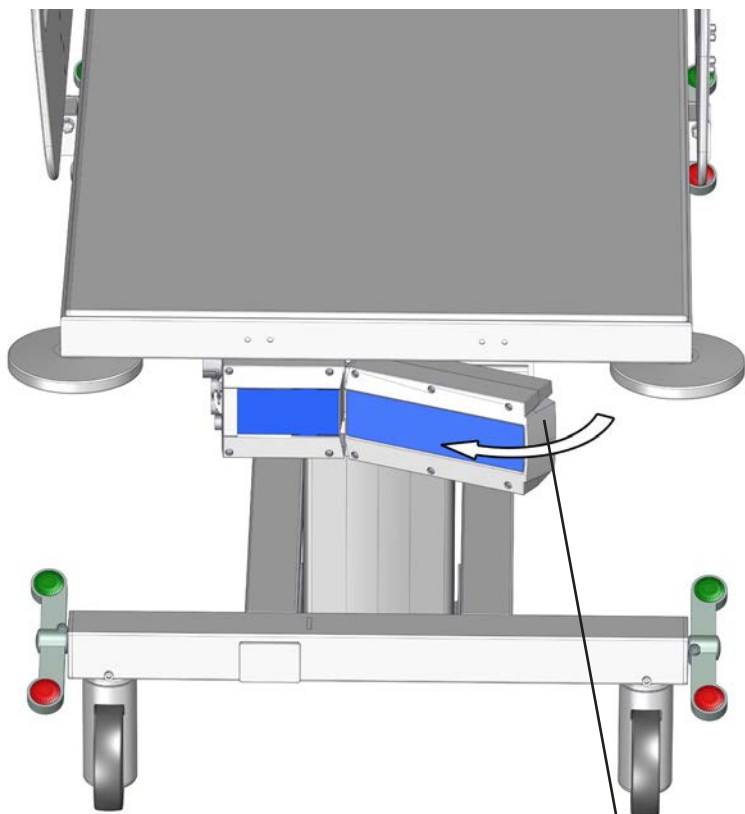
Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

Resetting

If the lifting function does not work properly or if the two lifting columns are out of sync, try resetting the lifting system. Press and hold the up button and the down button simultaneously. The columns will move down after a few seconds while a beep tone is heard. Keep holding both buttons until the beep tone disappears. The lifting columns are now reset.


Charging the battery

The battery can height adjust the change trolley up to 30 times when fully charged.


Release the lock to remove the battery for charging.

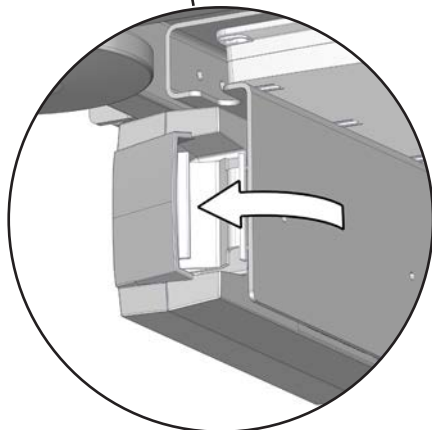Reattach the battery after charging before placing the patient on the change trolley.

en

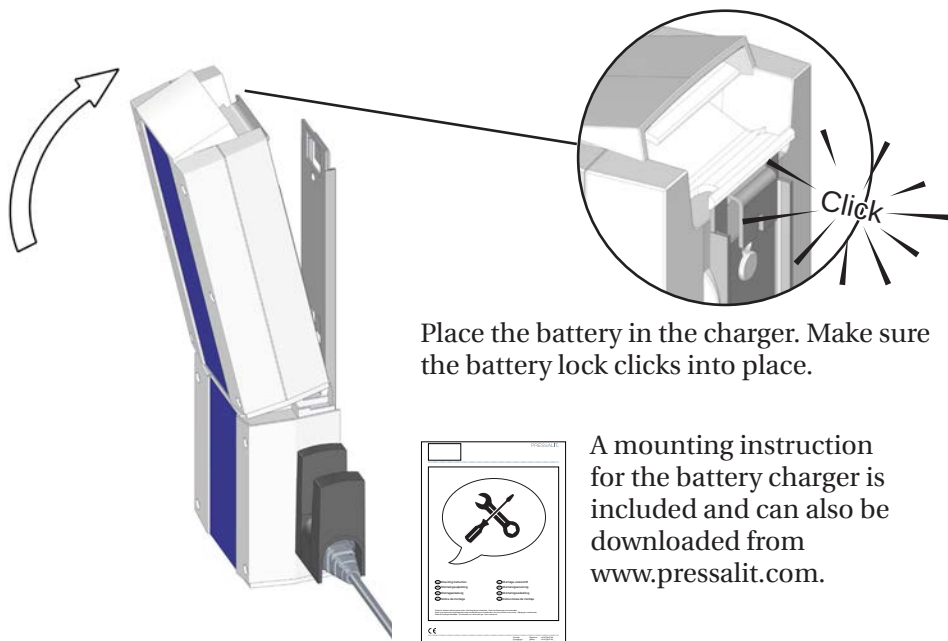Place the battery in the charger. Make sure the battery lock clicks into place.

A mounting instruction for the battery charger is included and can also be downloaded from www.pressalit.com.

CAUTION

When the battery is running low, a beep tone is heard when you operate the remote control. Although there is still enough power to raise or lower the unit, it is of utmost importance that you charge or replace the battery immediately. Failure to do so will soon cause the lifting function to stop working.

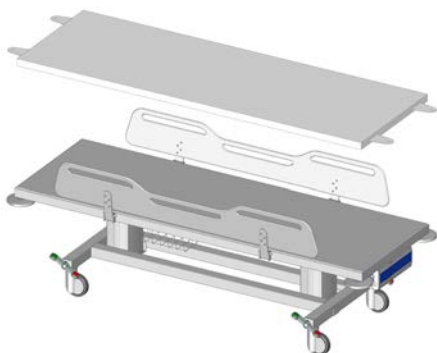
IMPORTANT

If the battery has not been used for more than a week, it enters into standby mode to save power and to protect the battery. Place the battery in the charger for approx. 15 seconds to exit the standby mode before use. Please note that there is no audio signal to indicate the standby mode or to indicate exit of standby mode. If there is still no power after exit of standby mode, the battery needs to be charged. The battery must be recharged every 6 months.


Accessories

Mattress

The optional mattress can be placed on the change trolley for optimal comfort. Both sides of the mattress can be used, as it is reversible.

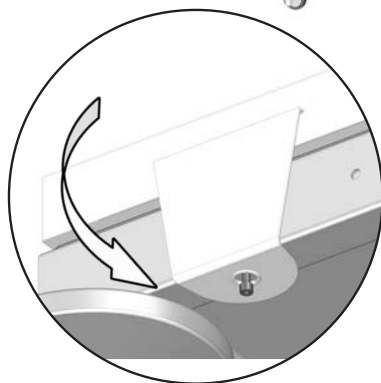Unscrew the locking screw for the mattress ① approximately one centimeter.

Pull the mattress flap down over the screw. Do the same with all four flaps.

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents/ disinfectants tested by Pressalit can be found on www.pressalit.com.

A mounting instruction is included with the mattress, and can also be downloaded from www.pressalit.com.

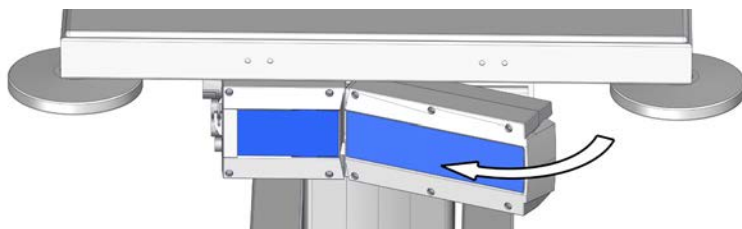
IMPORTANT

Check for damages on the mattress on a regular basis. This will prevent infection of the mattress.

Cleaning

Remove the battery before cleaning the change trolley.

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents/disinfectants tested by Pressalit can be found on www.pressalit.com.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

CAUTION

Do not perform any type of cleaning or disinfection of the change trolley while the patient is lying on it.

Electromagnetic compatibility

Electromagnetic immunity

Test standard	Compliance level	Test level / details	Test results
EN60601-1-2 and EN17966 (Home healthcare environment)	=> 10V/m 2,7GHz	15V/m Frequency range: 80MHz- 6GHz 1% step size	Passed
EN60601-1-2 (Proximity fields from RF wireless communications equipment)	=> 10V/m 2,7GHz	15V/m Frequency range: 385 - 5785 MHz	Passed
IEC 61000-4-4 (Electrical fast transients / bursts)		Quick pulse simulation	Passed
Conducted disturbances induced by RF	Pulse subjection, 6V	Pulse subjection, 10V	Passed
Electrostatic discharge		Statically charged person discharge on unit, 8kV and 15kV	Passed (*)

* If a statically charged person discharges on or near the shower trolley control box, a fault may occur in the lifting columns, and a reset must be performed. See "**Resetting**".

Electromagnetic emissions


Test standard	Test level / details	Test results
60601-1-2 ch. 7.1.7 - CISPR 14	30MHz – 1GHz	Passed

Technical data

Max user weight	200 kg / 440 lbs	
Max load	250 kg / 550 lbs	
Product weight	R94x2717	102 kg / 225 lbs
	R94x2721	106 kg / 234 lbs
Lifting height	550 – 950 mm (21 ⁵ / ₈ - 37 ³ / ₈)	
Battery capacity	2,25Ah	
Battery weight	1 kg (2.2 lbs)	
Battery charging time	3-4h	

Expected service life of the change trolley is 10 years.

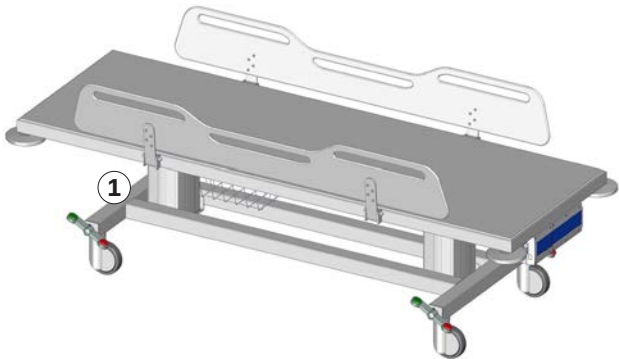en

Product label placement ①.

Trouble-shooting

Symptom	Cause	Action
No power.	Battery depleted.	Recharge the battery. See " Charging the battery ".
No power.	Battery in standby mode.	Place the battery in the charger for approx. 15 seconds. See " Charging the battery ".
The change trolley is skewed.	The lifting columns are out of sync.	Reset the lifting columns. See " Resetting ".
The trolley cannot be raised or lowered.	If a statically charged person discharges near the trolley control box, a fault may occur in the lifting columns.	Reset the lifting columns. See " Resetting ".

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Storage and transportation

- The temperature for transportation and storage of the unit must be between 5 and 40 degrees Celsius.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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