
R4950 - R4951
R4952 - R4953

SELECT wash basin lifter – Operation and maintenance manual	en
SELECT vaskeløfter – Brugs- og vedligeholdelsesvejledning	dk
SELECT Waschtisch-Lifter – Gebrauchs- und Pflegeanleitung	de
SELECT support de lavabo – Manuel d'utilisation et d'entretien	fr
SELECT muurframe voor wastafel – Gebruiks- en onderhoudshandleiding	nl
SELECT upphängning till tvättställ – Bruks- och underhållsanvisning	sv
SELECT oppheng til servant – Bruks- og vedlikeholdsveiledning	no
SELECT soporte de lavabo - Instrucciones de uso y mantenimiento	es
SELECT 洗面盆支架 - 操作和维护手册	中文

English	3
Dansk	18
Deutsch	33
Français	48
Nederlands	63
Svenska	78
Norsk	93
Español	107
中文	122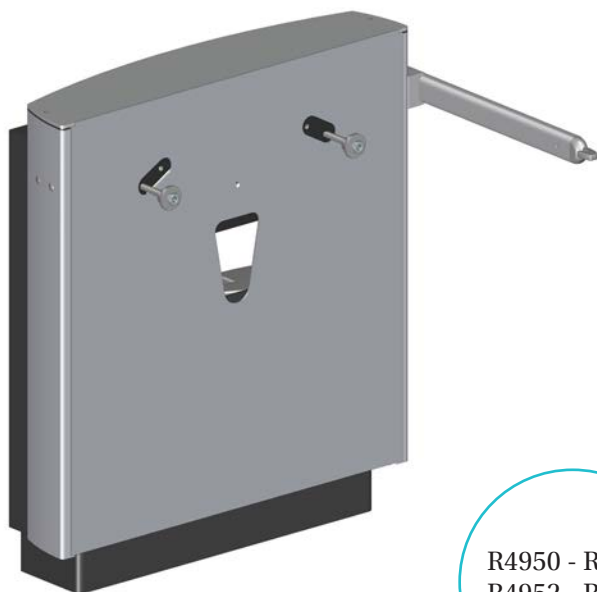

en	The guarantee covers faults or defects in material or manufacture within a period of 5 years. Products are subject to minor technical modifications and design deviations. E. & O.E.
dk	Garantien dækker i 5 år fra fakturadato for materiale- og fabrikationsfejl. Der tages forbehold for trykfejl, tekniske ændringer og modelafvigelser.
de	Die Garantie gilt für 5 Jahre ab Rechnungsdatum für Material- und Herstellungsfehler. Druckfehler, technische Änderungen und Modellabweichungen vorbehalten.
fr	La garantie couvre les vices de fabrication et de matériaux pour une durée de 5 ans à partir de la date de facturation. Sous réserve de modifications techniques et d'erreurs d'impression.
nl	De garantie is tot 5 jaar na factuurdatum geldig voor materiaal- en fabricagefouten. Onder voorbehoud van drukfouten, technische wijzigingen en modelafwijkingen.
sv	Garantin gäller i 5 år från fakturadatum för material- och fabrikationsfel. Med reservation för tryckfel, tekniska ändringar och modellavvikelser.
no	Garantien gjelder i 5 år fra fakturadato for material- og fabrikkasjonsfeil. Med forbehold for trykkfeil, tekniske endringer og modellforskjeller.
es	La garantía cubre fallos de material o fabricación durante los 5 años posteriores a partir de la fecha de factura. Mantenemos reservas en caso de problemas de presión, modificaciones técnicas y variaciones de modelo.
中文	保修涵盖 5 年内出现的材料或制造故障或缺陷。产品的技术和设计可能稍微发生变化，恕不另行通知。

Content

Symbols used in this manual	4
General safety	5
Product label	6
Intended use	8
Intended operator profile	8
Operating environment	8
Assembly and installation	9
Versions	10
Optional installation covers	10
Optional wash basins	11
Operating instructions	12
Safety stop function	13
To remove limescale	14
Maintenance	14
Service inspection	15
Technical data	16
Trouble-shooting	17
Disposal and recycling	18

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressaliticare.com.

en

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury
warning

Electricity or potentially
dangerous voltage.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.


CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label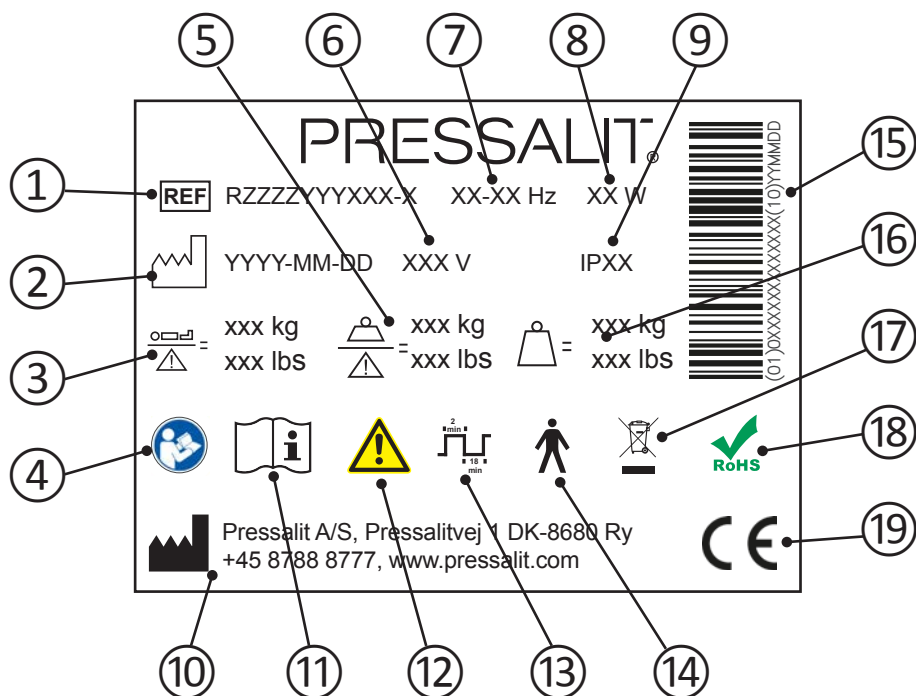


en

- 1 Article number
- 2 Production date
- 3 Maximum user weight
- 4 Consult manual for use
- 5 Safe working load
- 6 Mains voltage
- 7 Frequency

-
- ⑧ Maximum power consumption
-
- ⑨ IP class
-
- ⑩ Name and address of manufacturer
-
- ⑪ Consult manual for use
-
- ⑫ Consult the manual for important cautionary information such as warnings and precautions.
-
- ⑬ Duty cycle. The motor on height-adjustable products can be used for a maximum of 2 minutes of continuous operation, followed by 18 minutes pause.
-
- ⑭ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock.
-
- ⑮ GS1-128 barcode
-
- ⑯ Product weight
-
- ⑰ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
-
- ⑱ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
-
- ⑲ This product is CE-marked in accordance with the European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and with Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at www.pressalitcare.com.

Product label placement ①.

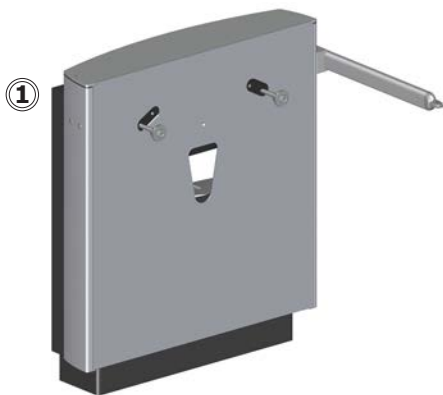Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room.

en The product is intended to be used at room temperature, in a stand-alone environment, in private homes, nursing homes and hospitals under normal working conditions.

CAUTION

Never place objects underneath the wash basin.

Versions

The wash basin lifter comes in four versions.

R4950

R4951

R4952

R4953

Optional installation covers

R2056

Fits wash basins R205x

R2057

Fits wash basins R202x

Optional wash basins

R2052

R2053


R2020

R2021


Operating instructions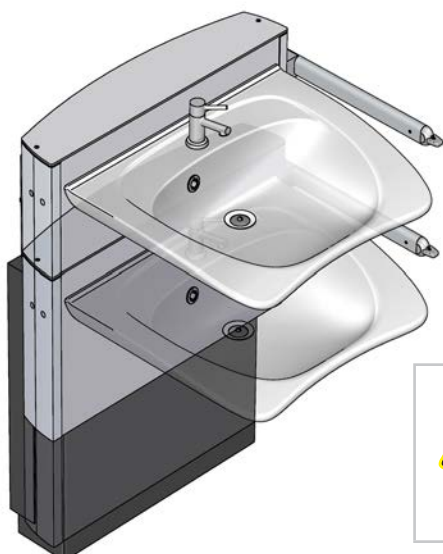

The wash basin lifter can be adjusted 300 mm up and down.
Max load is 150 kg.

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

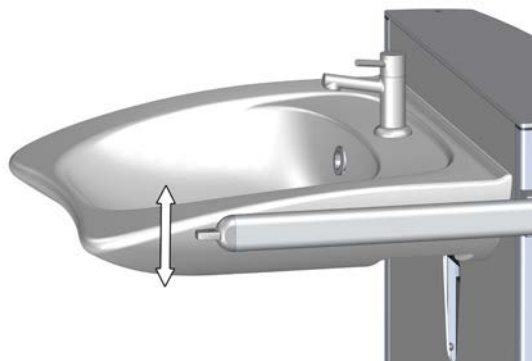R4950 and R4952

The height is adjusted by the lever control.
Press and hold the lever up to raise the wash basin, and press and hold the lever down to lower the basin. When the lever is released, the motion stops.


R4951 and R4953

The height adjustment is controlled by the wired hand control unit.

Press and hold the Up button to raise the wash basin, and press and hold the Down button to lower it. When the button is released, the motion stops.

Safety stop function

The wash basin lifters **R4952** and **R4953** are equipped with a safety stop function.

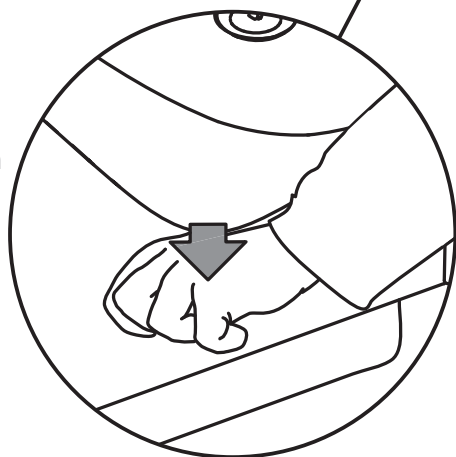If the wash basin comes in contact with an object when it is being lowered, the motion stops immediately.

If an object comes in contact with the underside of the wash basin when it is stationary, the safety function is activated and the wash basin is lifted.

The wash basin lifter can be used again as soon as the obstruction has been removed.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts of 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the wash basin lifter is in use.

Always read the material safety data sheet (MSDS) and supplied instructions for the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

Maintenance

Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that service inspection is done once a year.

Service inspection

Take the wash basin to its highest position.

Disconnect mains before removing the covers.

Remove the lower front cover ①. You do not need any tools, as the cover has a click lock.

Check all electrical connections for the control box ②.


Lubricate the inner square tubes ③ with WD-40 or equivalent. You have to reach up under the upper front cover to reach the tubes.

Reconnect mains.


Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

Reattach the lower front cover.

R4950 and R4951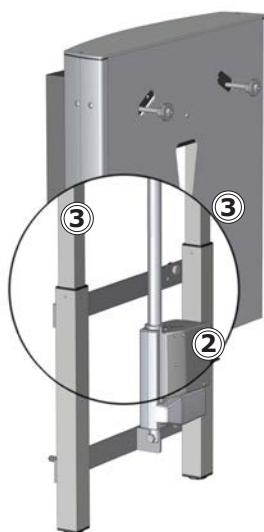

R4952 and R4953

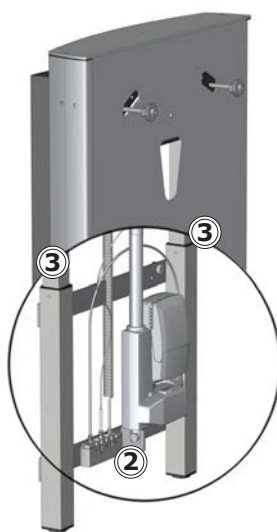Technical data

Max load	150 kg
Materials	Powder-laquered stainless steel Aluminium Plastics
Dimensions (width x height x depth)	666 mm x 750 mm x 170 mm
Weight	R4950, R4952: 28.5 kg
	R4951, R4953: 28 kg
IP class	IPX6
Power specifications	
Max consumption	75 W
Voltage	Mains connection: 230V/120V/100V
Frequency	50/60 Hz
Current	3.7 A

Trouble-shooting

Symptom	Cause	Action
The product will not work, even though the controls is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Polystyrene	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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